



Assembly Instructions on the Hill-Rom CareAssist® Bed  
Mizuho OSI Patient Helper P/N 3423

1. Attach Mizuho OSI Patient Helper Mounting Sleeve to the Hill-Rom CareAssist® Bracket using the provided hardware.
2. Insert the Patient Helper Upright into the Patient Helper Mounting Sleeve and secure with the Quick Release Knob.
3. Attach the Trapeze Arm Assembly and Heavy Duty Trapeze.
4. Adjust the Trapeze height as needed with the “S” hook on the chain.
5. Swivel and lock the Trapeze Arm Assembly in any position using the Swivel Lock Handle.

 **Failure to securely tighten all knobs may lead to product failure and injury to the patient.**

