



Assembly Instructions on the Stryker Secure II™ Bed
Mizuho OSI Patient Helper P/N 3423

1. Attach Mizuho OSI Patient Helper Mounting Sleeve to the Stryker Patient Helper Bracket using the provided hardware.
2. Insert the Patient Helper Upright into the Sleeve and secure with the Quick Release Knob.
3. Attach the Trapeze Arm Assembly and Heavy Duty Trapeze.
4. Adjust the Trapeze height as needed with the “S” hook on the chain.
5. Swivel and lock the Trapeze Arm Assembly in any position using the Swivel Lock Handle.

 **Failure to securely tighten all knobs may lead to product failure and injury to the patient.**

