



Assembly Instructions for a Single Traction Frame on the Stryker Secure II™ Bed
Mizuho OSI Frame P/N 3143

1. Insert two each 18-1/2" I.V. Post w/Clamp into the head end receptacles of the bed.
2. Connect the 18-1/2" I.V. Post w/Clamp with a 36" Plain Bar.
3. Attach a 31" Offset Double Clamp Bar (ODCB) to the 36" Plain Bar. The swivel end of the 31" ODCB will be on top and the offset will face toward the foot end.
4. Insert two each 13" Angled I.V. Post w/Clamp into the foot end receptacles of the bed.
5. Connect the 13" Angled I.V. Post w/Clamp with a 36" Plain Bar.
6. Attach a Cross Clamp to the center of the 36" Plain Bar facing away from the patient.
7. Insert a 48" Swivel Clamp Bar (SCB) into the Cross Clamp. Make sure the 48" SCB is aligned with the 31" ODCB at the head end.
8. Connect the 31" ODCB and the 48" SCB with an Adjustable Bar.
9. Hang the Deluxe Trapeze from the Adjustable Bar.

 **Failure to securely tighten all knobs may lead to product failure and injury to the patient.**

