



Assembly Instructions for a Balkan Traction Frame on the Hill-Rom TotalCare® Bed
Mizuho OSI Frame P/N 3243

1. Attach the Four Piece Traction Bracket Set as per installation instructions. This will be the foundation for the Balkan Frame.
2. Insert a 56" Single Clamp Bar with Post into the Foot-End Bracket. Repeat on other side.
3. Insert a 56" Offset Single Clamp Bar with Post into the Head-End Bracket. Repeat on other side.
4. Connect the Head and Foot-End Single Clamp Bars with two 74" Plain Bars.
5. Starting close to the foot-end, connect the two 74" Plain Bars with a 35" Double Clamp Bar. Make sure the knobs face down for easier access. Repeat the process in the middle of the bed and at the head-end. The middle bar will be used to capture the Deluxe Trapeze.
6. Attach two each Cross Clamps to the offset portion of the head-end Single Clamp Bars w/Post. Make sure the Cross Clamps are at the same height on each bar.
7. Connect the two Cross Clamps with a 42" Plain Bar.
8. Connect the Deluxe Trapeze to the middle 35" Double Clamp Bar.
9. Circle the perimeter of the bed and tighten all knobs securely.

 **Failure to securely tighten all knobs may lead to product failure and injury to the patient.**

