



Assembly Instructions for a Balkan Traction Frame on the Stryker Secure II™ Bed
Mizuho OSI Frame P/N 3236

1. Insert two each P/N 3050 18-1/2" I.V. Post W/Clamp into the head end receptacles of the bed.
2. Connect the 18-1/2" I.V. Post W/Clamp with P/N 3003 36" Plain Bar.
3. Attach two each P/N 3030 31" Offset Double Clamp Bars (ODCB) to the 36" Plain Bar. The swivel end of the ODCB is on top and the offset will face toward the foot end of the bed.
4. Insert two each P/N 3075 13" Angled I.V. Post W/Clamp into the foot end receptacles of the bed.
5. Connect the 13" Angled I.V. Post W/Clamp with P/N 3003 36" Plain Bar.
6. Attach two each P/N 3033 Cross Clamps to the 36" Plain Bar.
7. Insert two each P/N 3022 48" Swivel Clamp Bars (SCB) into the Cross Clamps. The 48" SCB must align with the 31" ODCB at the head end. These will capture the Adjustable Bars being used for the top rails.
8. Connect the 31" ODCB and the 48" SCB with 2 each P/N 3000 Adjustable Bars.
9. Using six Cross Clamps and three 36" Plain Bars, connect the Adjustable Bars as shown.
10. Hang P/N 3071 Deluxe Trapeze from the middle 36" Plain Bar.

 **Failure to securely tighten all knobs may lead to product failure and injury to the patient.**

