



# Hana<sup>®</sup> Setup Checklists



Users must read and understand the Hana Orthopedic Surgery Table Owner's Manual (NW0508) before using this device. Instructions provided are for suggested setups. Final determination of patient positioning and setup is at the discretion of the surgeon and surgical team.

**Table Setup Checklist must be completed prior to completing the other checklists.**

## Table Setup Checklist

Performance Criteria	Manual Section	Completed
Identify patient weight capacity and label location	3.2	<input type="checkbox"/>
Demonstrate connecting the Power Cord to the table	3.2.1	<input type="checkbox"/>
Demonstrate connecting the Hand Pendant	3.3	<input type="checkbox"/>
Identify the power switch and turn on the table	3.2.1	<input type="checkbox"/>
Identify the Control Pad	3.2.2	<input type="checkbox"/>
Identify the Emergency Stop Button	3.4	<input type="checkbox"/>
Complete a Function Check	5.0	<input type="checkbox"/>
Demonstrate locking the casters	3.5	<input type="checkbox"/>
Demonstrate inserting and removing a Leg Spar (left or right) into the appropriate Spar Mount	6.1.1	<input type="checkbox"/>
Demonstrate adjusting the Leg Spar(s) using the Leg Spar Controls	6.1.2	<input type="checkbox"/>
Demonstrate sliding the Patient Safety Strap around the Hana Table and the patient	6.2	<input type="checkbox"/>
Demonstrate installing the Hana Arm Boards	6.7	<input type="checkbox"/>
Explain how to put the Traction Boots on the patient's feet and demonstrate securing the boots to the table	6.4	<input type="checkbox"/>
Explain the use of the Traction Hook Extenders	6.5	<input type="checkbox"/>

Notes:

Employee Name


Signature

Date

Observer Name

Signature

Date

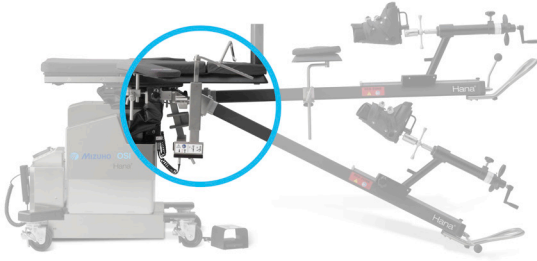

Hana® Table Setup Checklist must be completed prior to completing this checklist.

## Femur Lift and Femoral Hook Installation Checklist

Performance Criteria	Manual Section	Completed
Demonstrate installing and removing the Jack Mount Assembly	6.8.1, 6.8.2	<input type="checkbox"/>
Demonstrate attaching the Femur Lift Assembly to the two Femur Lift Expansion Junction Boxes	6.9	<input type="checkbox"/>
Demonstrate connecting the Coiled Hook Motor Cables to the Expansion Junction Box and the Femur Lift	6.9.1, 6.9.2	<input type="checkbox"/>
Demonstrate mounting the Femur Lift to the Jack Mount Assembly	6.9.2	<input type="checkbox"/>
Show where and how to plug in the Femur Lift Foot Pedal	6.9.4	<input type="checkbox"/>
Explain the use of the Femoral Hook and Femoral Hook Support	6.10	<input type="checkbox"/>
Explain the appropriate cleaning method for the washing, decontamination, and sterilization procedures before each use of Mizuho OSI's Femoral Hooks and Femoral Hook Support	6.10, 10	<input type="checkbox"/>

Notes:

\_\_\_\_\_  
Employee Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date

\_\_\_\_\_  
Observer Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date



Hana® Table Setup Checklist must be completed prior to completing this checklist. Refer to Owner's Manual **Section 8.8** for additional hip pinning setup details.

## Hip Pinning: Supine with Bilateral Skin Traction Checklist

Performance Criteria	Manual Section	Completed
Demonstrate installing and removing the Jack Mount Assembly	6.8.1, 6.8.2	<input type="checkbox"/>
Demonstrate attaching the Left and Right Leg Spars	8.8	<input type="checkbox"/>
Demonstrate placing the Pelvic Pad on table	6.3	<input type="checkbox"/>
<b>Optional:</b> Demonstrate attaching and removing the Patient Transfer Board	9.3.1, 9.3.2	<input type="checkbox"/>
Demonstrate sliding the Patient Safety Strap into place	6.2	<input type="checkbox"/>
Explain how to put the Traction Boots on the patient's feet	6.4	<input type="checkbox"/>
Explain patient transfer process onto the table in supine position	8.8	<input type="checkbox"/>
Demonstrate inserting the padded Perineal Post onto the table	6.3	<input type="checkbox"/>
Demonstrate attaching and locking the Traction Boots to the Leg Spar Traction Mounts	6.4	<input type="checkbox"/>
Demonstrate installing Hana Arm Boards	6.7	<input type="checkbox"/>
<b>Optional:</b> Demonstrate installing Cross Arm Support using Clark Socket	9.6	<input type="checkbox"/>
<b>Optional:</b> Demonstrate attaching the Lower Leg Support to the non-operative Leg Spar	9.7	<input type="checkbox"/>
<b>Optional:</b> Demonstrate installing the Head-End and Foot-End Drape Rod Assemblies and suspending the Patient Isolation Drape	9.10	<input type="checkbox"/>

Notes:

\_\_\_\_\_  
Employee Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date

\_\_\_\_\_  
Observer Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date



Hana® Table Setup and Femur Lift and Femoral Hook Installation Checklists must be completed prior to completing this checklist. Refer to Owner’s Manual **Section 8.9** for additional AATHA setup details.

## Anterior Approach Total Hip Arthroplasty Checklist

Performance Criteria	Manual Section	Completed
Demonstrate installing and removing the Jack Mount Assembly	6.8.1, 6.8.2	<input type="checkbox"/>
Demonstrate attaching the Left and Right Leg Spars	8.9	<input type="checkbox"/>
Demonstrate placing the Pelvic Pad on table	6.3	<input type="checkbox"/>
Demonstrate attaching the Femur Lift Assembly to the two Femur Lift Expansion Junction Boxes (left and right)	6.9	<input type="checkbox"/>
Demonstrate connecting the Coiled Hook Motor Cables to the Expansion Junction Box & the Femur Lift	6.9.1, 6.9.2	<input type="checkbox"/>
Demonstrate mounting the Femur Lift to the Jack Mount Assembly	6.9.2	<input type="checkbox"/>
Show where and how to plug in the Femur Lift Foot Pedal	6.9.4	<input type="checkbox"/>
<b>Optional:</b> Demonstrate attaching and removing the Patient Transfer Board	9.3.1, 9.3.2	<input type="checkbox"/>
Explain how to put the Traction Boots on the patient’s feet	6.4	<input type="checkbox"/>
Explain patient transfer process onto the table in supine position	8.9	<input type="checkbox"/>
Demonstrate inserting the padded Perineal Post onto the table	6.3	<input type="checkbox"/>
Demonstrate attaching and locking the Traction Boots to the Leg Spar Traction Mounts	6.4	<input type="checkbox"/>
Demonstrate installing the Hana Arm Boards	6.7	<input type="checkbox"/>
<b>Optional:</b> Demonstrate installing Cross Arm Support using Clark Socket	9.6	<input type="checkbox"/>
<b>Optional:</b> Demonstrate attaching the Lower Leg Support to the non-operative Spar	9.7	<input type="checkbox"/>
Explain the use of the Femoral Hook and Femoral Hook Support	6.10	<input type="checkbox"/>
Demonstrate attaching the Femoral Hook Support	6.10	<input type="checkbox"/>
Demonstrate inserting the Femoral Hook into Femoral Hook Support	6.10	<input type="checkbox"/>

Notes:

\_\_\_\_\_  
Employee Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date

\_\_\_\_\_  
Observer Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date



Hana® Table Setup Checklist must be completed prior to completing this checklist.  
Refer to Owner's Manual **Section 8.10** for additional hip arthroscopy setup details.

## Hip Arthroscopy Checklist

Performance Criteria	Manual Section	Completed
Demonstrate installing and removing the Jack Mount Assembly	6.8.1, 6.8.2	<input type="checkbox"/>
Demonstrate attaching the Left and Right Leg Spars	8.10	<input type="checkbox"/>
Demonstrate placing the Pelvic Pad on table	6.3	<input type="checkbox"/>
Demonstrate sliding the Patient Safety Strap into place	6.2	<input type="checkbox"/>
<b>Optional:</b> Demonstrate attaching and removing the Patient Transfer Board	9.3.1, 9.3.2	<input type="checkbox"/>
Explain how to put the Traction Boots on the patient's feet	6.4	<input type="checkbox"/>
Explain patient transfer process onto the table in supine position	8.10	<input type="checkbox"/>
Demonstrate inserting the large padded Perineal Post onto the table	6.3	<input type="checkbox"/>
Demonstrate attaching and locking the Traction Boots to the Leg Spar Traction Mounts	6.4	<input type="checkbox"/>
Demonstrate installing Hana Arm Boards	6.7	<input type="checkbox"/>
<b>Optional:</b> Demonstrate installing Cross Arm Support using Clark Socket	9.6	<input type="checkbox"/>
<b>Optional:</b> Demonstrate attaching the Lower Leg Support to the non-operative Spar	9.7	<input type="checkbox"/>

Notes:

\_\_\_\_\_  
Employee Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date

\_\_\_\_\_  
Observer Name

\_\_\_\_\_  
Signature

\_\_\_\_\_  
Date