



ProAxis® Table Setup Checklist must be completed prior to completing this checklist.

Prone Patient Positioning Setup Checklist

Performance Criteria	Manual Section	Completed
Identify the Patient Care Kit components and explain preparation steps	7.2	<input type="checkbox"/>
Demonstrate setup of the ProneView Cushion Insert and Helmet	8.1	<input type="checkbox"/>
Install the Rail Mounts for the Torso Trolley	8.1	<input type="checkbox"/>
Connect the Cable Connectors of the Rail Mounts to the corresponding ports	8.1	<input type="checkbox"/>
Demonstrate installing the (Torso Trolley) Chest Tray	8.1	<input type="checkbox"/>
Demonstrate attaching the Chest Pad onto the Chest Tray	8.1	<input type="checkbox"/>
Demonstrate installing the Arm Board Mounting Brackets	8.1	<input type="checkbox"/>
Demonstrate attaching the Articulating Arm Boards	8.1	<input type="checkbox"/>
Demonstrate the functionality of the Articulating Arm Boards	8.1	<input type="checkbox"/>
Demonstrate installing and positioning the Combination Hip and Thigh Pads	8.1	<input type="checkbox"/>
Demonstrate attaching the Leg Boards	8.1	<input type="checkbox"/>
Demonstrate attaching the Leg Sling	8.1	<input type="checkbox"/>
Demonstrate mounting the Retractor Adapter	8.1	<input type="checkbox"/>
Explain the use of the Retractor Adapter	8.1	<input type="checkbox"/>
Demonstrate placement of the Head Support on the Torso Trolley	8.1	<input type="checkbox"/>
Explain how to place Patient Care Kit Covers on pads	8.1	<input type="checkbox"/>
Explain how to log roll the patient to a prone position	8.1	<input type="checkbox"/>
Identify patient anatomical landmarks that align with support pads	8.1	<input type="checkbox"/>
Demonstrate application of buttocks strap and safety strap	8.1	<input type="checkbox"/>
Describe the use of the Mizuho OSI Tempur-Pedic Medical Pillows	8.1	<input type="checkbox"/>

Notes:

Employee Name

Signature

Date

Observer Name

Signature

Date



ProAxis® Table Setup Checklist must be completed prior to completing this checklist.

Lateral Patient Positioning Setup Checklist

Performance Criteria	Manual Section	Completed
Demonstrate installing the Head-End Supine Top	8.4	<input type="checkbox"/>
Connect the Cable Connector to the port	8.4	<input type="checkbox"/>
Demonstrate installing the Foot-End Supine Top	8.4	<input type="checkbox"/>
Attach the Mizuho OSI Tempur-Pedic Medical Pad	8.4	<input type="checkbox"/>
Explain how to complete patient transfer in supine position	8.4	<input type="checkbox"/>
Explain how to place and secure patient in the lateral position	8.4	<input type="checkbox"/>
Demonstrate installing the Universal Side Rail Adapter	8.4	<input type="checkbox"/>
Explain the use of the Universal Side Rail Adapter	8.4	<input type="checkbox"/>
Demonstrate attachment of safety strap	8.4	<input type="checkbox"/>
Describe the use of the Mizuho OSI Tempur-Pedic Medical Pillows	8.4	<input type="checkbox"/>
Optional: Demonstrate attaching the Clark Socket onto Universal Side Rail Adapter	8.4	<input type="checkbox"/>
Optional: Demonstrate attaching the Aluminum Side Rail Socket onto Universal Side Rail Adapter	8.4	<input type="checkbox"/>
Optional: Demonstrate installing the Lateral Arm Board Unit into the Clark Socket	8.4	<input type="checkbox"/>
Optional: Demonstrate mounting the Lateral Positioner Set or Deluxe Lateral Positioner Set	8.4	<input type="checkbox"/>

Notes:

Employee Name

Signature

Date

Observer Name

Signature

Date