

HANA® STERILE DRAPING TECHNIQUE INSTRUCTIONS FOR USE

As recommended by Joel Matta, MD

REF 6855SDK-1 **CONT** 1

INTENDED USE

This draping technique is intended to be used with the Hana® Surgical Table for Anterior Approach Total Hip Arthroplasty.



CAUTION: This Sterile Drape Kit is SINGLE USE ONLY. Discard after use. Reuse of any Sterile Drape will result in a non-sterile field and cross-contamination.

Hana Sterile Drape Kit 6855SDK-1 includes:

- (a) 70 x 100" Clear Rectangle Drape (2)
- (b) Clear Lower U-Drape (1)
- (c) 6855AUD-1, Upper U-Drape w/Femur Lift Cover (1)

NOTE: Position the patient on the surgical table as described in the Hana® Orthopedic Surgery Table Owner's Manual. Prior to prepping the surgical site, small non-sterile upper and lower U-Drapes (not included in this drape kit) may be applied to frame the surgical site.

1. Apply a Clear Rectangle Drape (a) over the patient's knees and over the end of the Hana Leg Spars (Figure 1).
2. Apply the second Clear Rectangle Drape (a) over the patient's upper body, covering both arms positioned on the arm boards. Unfold headwards (Figure 2).
3. Use the adhesive edges of the Clear Lower U-Drape (b) to frame the lower half of the surgical window (Figure 3). The lower portion of this drape extends over the legs. The upper portion of this drape may be tucked behind the Hana Femur Lift (Figure 4).
4. Place the Upper U-Drape (c). Refer to Instructions for Use printed on the 6855AUD-1 drape package.
 - a. Use the adhesive edges of the Upper U-Drape to frame the upper portion of the surgical site (Figure 5).
 - b. Extend the Upper U-Drape toward the head. This may be used to create the barrier between anesthesia and the sterile field (Figure 6).
 - c. Use the integrated Femur Lift Cover of the Upper U-Drape to drape the Femur Lift (Figure 7).
 - d. Use the Integrated Clear Flap of the Upper U-Drape to drape the Femoral Hook Support (Figure 8).
5. Apply a universal fluid pouch (not included), if desired, to collect irrigation fluids.
6. Apply incise drape (not included), as preferred by the surgeon.

RE-ORDER 6855SDK-6, 6 KITS PER CASE



www.mizuhos.com/resource/symbols/



Mizuho OSI • 30031 Ahern Ave. • Union City, CA 94587-1234 USA • (800) 777-4674
• +1 (510) 429-1500 • FAX +1 (510) 429-8500 • www.mizuhos.com

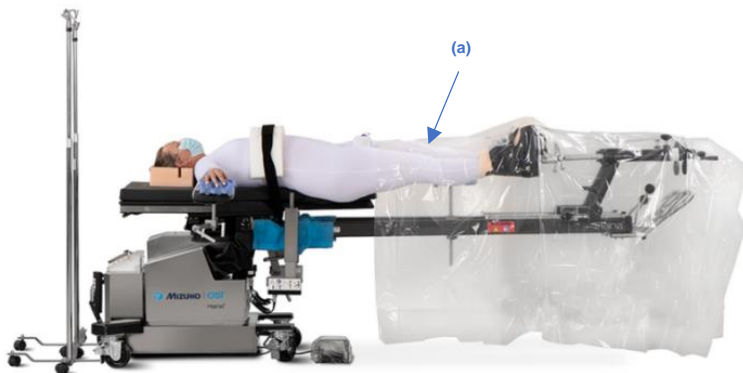


Figure 1



Figure 2



Figure 3



Figure 4



Figure 5

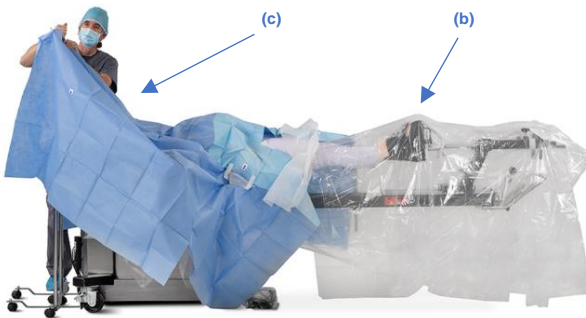


Figure 6

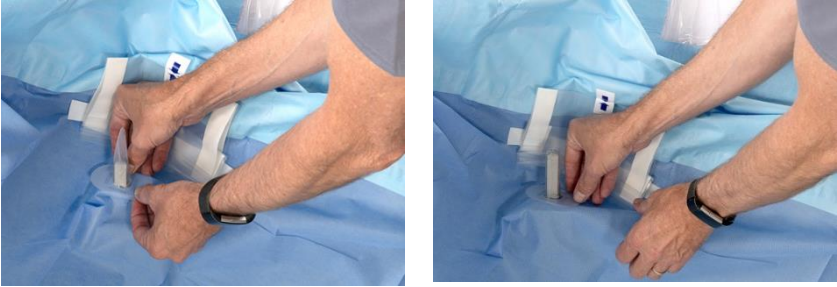


Figure 7



Figure 8